



Application Card



How to Use

Convatec Little Ones® Adhesive Coupling Technology™ system*


Baseplate Preparation



1 **Measure** the stoma size using the measuring guide provided.



2 **Trace** the stoma outline you have selected onto the release paper on the baseplate. The small starter hole should be in the centre of your tracing.

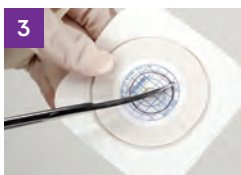

3 **Cut** the centre hole the same size as the stoma. Cut along the tracing with curved, short-blade scissors.



4 Remove the release paper on the baseplate.

Application of Baseplate



5 Apply the baseplate around the stoma, holding for 30 seconds to ensure a good seal is achieved.



6 If you are using a flexible baseplate (taped border), you can remove the paper on one side of the baseplate and smooth onto the skin. Repeat with the other side, making sure the tape is smooth and wrinkle-free.


Attachment of Pouch-to-Baseplate



7 If you are using a drainable pouch, it is recommended that you close the InvisiClose® clipless tail closure system by folding upward and pressing the interlocking closures together until you feel them lock into place. A "click" indicates a secure closure.



8 Make sure the inner surfaces of the pouch are separated and that the pouch contains a small amount of air.

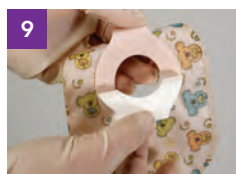

9 Remove the release paper from the ring on the pouch.



10 Line up the bottom half of the adhesive ring with the base of the clear landing zone on the baseplate. Once in the correct position, roll your fingers over the upper portion to secure the pouch. Always look beneath the top of the pouch at the adhesive to ensure the pouch-to-baseplate adhesion is secure.



Pouch Repositioning and Removal



10 To reposition the pouch, peel the pouch from the clear square "landing zone" by holding the tab on the pouch ring with one hand and the end of the landing zone with the other hand.

Then, peel the pouch away in a rolling motion and reposition as needed. Once ready to remove the pouch, have a some water and some soft cloths ready to clean the skin. To release the pouch from the skin, go slowly and gently, removing the baseplate and pouch together. Discard as necessary.

Application Card

InvisiClose®
Tail Closure System

How to Use InvisiClose® Tail Closure System

These instructions apply to the Little Ones® Adhesive Coupling Technology™ system

Directions to Close the Pouch



The InvisiClose® Tail Closure System is comprised of a **security flap (A)**, **inter-locking closures (B)**, and **outlet end-strips (C)**.



Hold the security flap against the front of the pouch revealing the inter-locking closures.



Fold the tail end up towards four times and ensure the inter-locking closures overlap.




Press the inter-locking closures together until you feel them lock (**click**) into place along their entire length.



Wrap the security flap around to the back of the pouch (**body side**) and press the inter-locking closures together until you feel them lock (**click**) into place.

Directions for Draining the Pouch

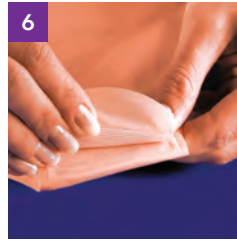

Turn the back of the pouch (body side) upwards. Carefully lift the security flap upwards to unlock.



While holding the security flap up against the front of the pouch, unlock the inter-locking closure by lifting at the corner and unraveling the folds four times.



To facilitate draining and/or cleaning, you may open the tail end by pushing in both ends of the outlet end-strips with your thumb and forefinger. If desired, pouch tail may be cleaned with water. Pat dry.

Tips:

- Remember to empty your child's pouch when it is one-third full.
- The use of soaps or cleansers in the pouch is not recommended. If desired, rinse only with tepid water.

For more information or if you have questions, call

0800 282 254

or visit our website at www.convatec.com/en-gb

